

Cleaning Instructions

for D001-I and D001-I-atc

1. Remove the tray from the organizer by gently pushing back the tab at the top of the tray.
2. Detach the clear cover by firmly pulling the cover off of the knobs on the sides of the organizer.
3. Pull out the accessory drawer firmly.
4. Wipe down each part with a cloth moistened with disinfectant fluid.
5. Replace the parts in reverse order.
6. Make sure to secure the tray under the tab at rear of the organizer.
7. Close the protective cover to help prevent cross-contamination.

Sterilizing Instructions

for D001-I-atc ONLY

1. Remove the tray from the organizer by gently pushing back the tab at the top of the tray.
2. Place the tray, including the burs, into the autoclave.
3. Run the autoclave at the preset cycles (134°C for 5 minutes or 121°C for 20 minutes).
4. After the completion of the sterilization cycle, return the tray to the organizer.
5. Make sure to secure the tray under the tab at rear of the organizer.
6. Close the protective cover to help prevent cross-contamination.


Developed by Israeli dentists. All materials are medical grade.

Manufactured by MedicNRG Ltd.

Fax: +972-4-675478, email: info@medicnrg.com,
www.medicnrg.com


MK-ZUPAN83

Panoramic Burr Organizer

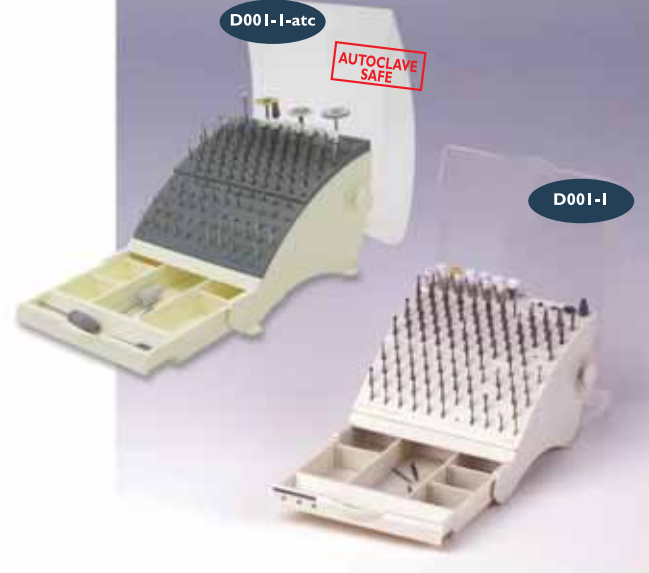

Panoramic stand for burs
Available in regular or autoclave safe models

Optimizing Your Endo Performance

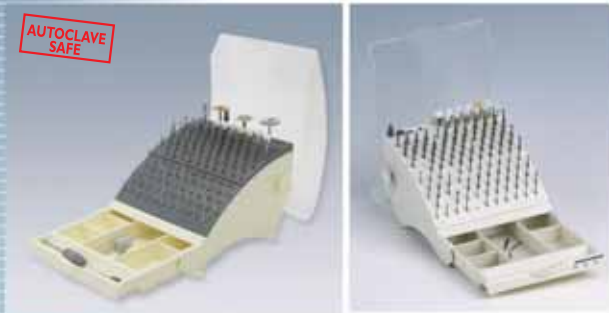

- Clear view and easy access to each individual burr
- Designated and marked shafts for the different burrs
- Removable tray, suitable for sterilization in the autoclave (D001-I-atc only)
- Adjustable accessory drawer
- Ventilated burr shafts and cradle
- Detachable protective cover
- Non-skid pads

Convenient Access

The panoramic shape and staggered shaft configuration allow a clear view and easy access to each burr.

Cross Contamination Avoidance

The burr tray is suitable for sterilization in the autoclave (D001-I-atc only). This, combined with the detachable clear cover helps prevent cross contamination and protects burrs.

Labeled Work-Station

The tray is clearly marked and separated into areas designated for the different types of burrs. In addition, the organizer includes a convenient accessory drawer with adjustable compartments for wedges, polishing discs, matrices, etc.

Ventilated Burr Shafts

The patented cradle and open back design allow drainage and ventilation of the burr shafts, which speed up drying time and prevents corrosion.

Easy Maintenance

The organizer is easily dismantled. Parts may be cleaned individually.

