

Tel Aviv, 14 September 2010

To: Mr. Arik Becker, CEO Medic NRG

Subject: Clinical Evaluation of ApexNRG-Rider Apex Locator

Dear Mr. Becker,

I have been using your "Rider" apex locator since July, 2010. I use it in my clinic, assembled on my Sybron (Analytic) micro-motor, both of which I take with me.

I have tested the device in hundreds of cases. I must admit, it took me 5-6 root canal treatments to adjust to the quality of the information displayed. But, since I have gotten used to the feedback, it has significantly changed my work methods when performing root canal treatments.

For example, I usually use the hand files only for the first steps of the procedure; I no longer need to pause the treatment to make a verification measurement using my old apex locator as I used to work before with or by taking an X-ray; and I even require less assistance from my dental assistant, as I do not perform all of these intermediate steps.

During the first procedures in which I tested the Rider, I performed a series of manual measurements to verify the accuracy of the Rider apex locator when used with a micro-motor, in addition to taking X-ray images. I was astonished by the device's level of accuracy. The unique and most important feature I find in the Rider apex locator is its ability to display the file's movements in real time. I know that I can trust the device to advise the exact location of the file at any moment, allowing me to finish the canal preparation using NiTi files all the way to the apical constriction.

The Rider apex locator's significant added value, which I definitely recognize now that I have confidence and experience when using the device, is that it is a huge time-saver. It has cut down canal treatment time by about 30%, compared to the time I invested before using the Rider. In addition, the quality of the canal preparation, due to the continuous use of the NiTi files is better and more uniform.

I have no doubt that as a result of the Rider's double function as both a manual apex locator and compatibility with handpieces, a smart sales campaign will eventually position this device as a "must-have" in every dental clinic that performs endo work, in addition to changing work habits of dental practitioners in root canal preparation.

Sincerely,

Dr. Moshe Zuckerman, DMD, B.Sc



רחוב נורדאו 29 פתח-תקוה, טל: 03-9305566
www.rozmedics.co.il

