




OP-2UMN034

NEW

MEDICNRG™
Precise Endo Technology

**Instructions for use
Mini File Holder**

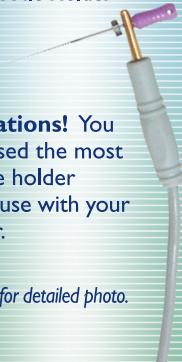

Please note: the mini file holder
is not supplied in a sterile state!

Manufactured by MedicNRG Ltd., Israel
Website: www.medicnrg.com
Fax: +972-4-675-4278


Congratulations! You
have purchased the most
advanced file holder
available for use with your
apex locator.

** See other side for detailed photo.*



- Insert the mini file holder (2) into the socket at the tip of the cable (1).

Please note: the file holder should be inserted into the cable in one direction only - the slit opening for the file should be facing outwards, and not inside the socket.

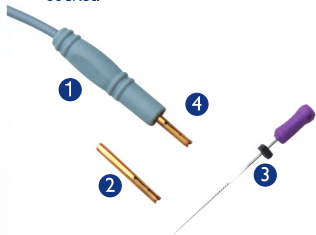

- File sizes thicker than 0.2, should be slipped into the first notch. Inserting a large file into the second groove may widen the file holder and thin files may slip out. Thin files may be inserted into the second notch.

- Insert your file (3) (either NiTi or regular) into the slit opening at the tip of the mini-file holder (2) until it reaches the groove (4).

The file is now secure in the holder.

- The mini file holder should be appropriately sterilized after each use, and between patients.

Please note: The mini file holder is compatible with most endo handpieces