



## Dear Customer

Your **File Organizer set** is suitable for both regular files and NiTi files.

The **File Organizer set** can be sterilized in the Autoclave. Please follow these directions precisely:

### Procedure:

When placing the **File Organizer set** into the Autoclave, open the main cover of the cylinder container ① and place the six small covers inside the main cover ② (see picture A). Place the two container parts on a sterilization sleeve or on a paper towel. This prevents direct contact

between the metal tray and the plastic container. When the sterilization process is completed, hold ② the main cover with the six cylinder covers inside on an angle facing the container with the six file organizer cylinders ① (see picture B). Securely close the main cover (see picture C).

### The File Organizer set is now sterile.

To use your File Organizer Set, remove only the main cover. The six inner cylinders remain securely closed and sterile (See picture D).



The File Organizer while in use. One inner cylinder set is open; the others are securely closed and sterile.

### **Sterilization conditions in the Autoclave:**

- Maximum Temperature: 134°c
- Sterilizing Time: 5 minutes
- Drying time: 10-15 minutes in Autoclave

To guarantee your continued enjoyment of the **File Organizer set** you must follow these directions precisely. Temperatures higher than 134°c may cause damage to the plastic.

Sincerely, **MedicNRG**

Design Pending

**MEDICNRG™**  
*Precise Endo Technology*

### **Autoclavable File Organizer Set**



**Instructions for sterilizing  
the File Organizer Set in the Autoclave**

**Optimizing Your Endo Performance**